CS

Gel

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INDICATION HYDROCYN aqua Gel is intended for moistening and debridement of acute and chronic wounds such as stage I-IV ulcers, venous stasis and diabetic ulcerations, post- surgery wounds, burns (1st - 2nd degree) and pressure ulcers.

DIRECTION FOR USE

- Debride wound, if necessary or cleanse wound with a HYDROCYN aqua Solution.
- Apply a generous amount (1/4" to 1/2" thick) of HYDROCYN aqua Gel to entire wound bed.
- Apply a thin coating to peri-wound skin area and allow HYDROCYN aqua Gel to dry.
- Cover with appropriate wound dressing or covering.
- Change dressing once a day or as indicated by the clinical condition of the wound. Maintain a moist wound environment between dressing changes.

STORAGE CONDITION HYDROCYN aqua Gel should be stored in a cool dry place (recommended storage temperature is 4 °C to 30 °C) away from direct sunlight/extreme weather condition, in its original packaging. After use, the tube should be closed again, before storing.

STABILITY Product is stable for 18 months if unopened and in its original packaging. Use prior to expiration date as stated on the packaging label. There is a shelf-life of 90 days after opening as long as it is closed immediately after use and protected from contamination or direct contact.

CAUTION Do not use the product if the tamper proof seal is damaged or tampered with. Packaging has to be discarded if signs of contaminants are seen. Do not dilute or use the product in combination with other topical cream unless with the advice of a health care specialist.

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RISK OF RE-USE HYDROCYN aqua Gel cannot be re-used if it is dispensed out from the tube.

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EC REP OBELIS S.A.
! Bactiguard AB