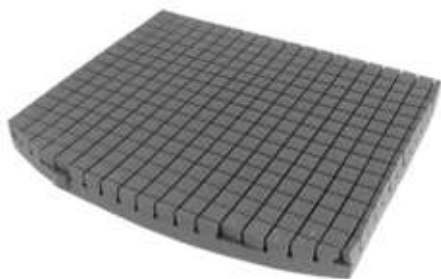
Evolight

DE	Gebruiksaanweisung	3
EN	Instructions for use	11
FR	Instructions d'utilisation	20
IT	Istruzioni per l'uso	30
ES	Instrucciones de uso	39
PT	Manual de utilização	47
NL	Gebruiksaanwijzing	56
SV	Bruksanvisning	66
DA	Brugsanvisning	74
NO	Bruksanvisning	83
FI	Käyttöohje	91
PL	Instrukcja użytkowania	100
HU	Használati utasítás	110
CS	Návod k použití	118
HR	Upute za uporabu	127
SL	Navodila za uporabo	136
SK	Návod na používanie	145
TR	Kullanma talimatı	154
EL	Οδηγίες χρήσης	163
RU	Руководство по применению	173
JA	取扱説明書	183
ZH	使用说明书	191
ZH^{HK}	使用說明書	199
KO	사용 설명서	207

1

2

3

INFORMATION

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INFORMATION

Date of last update: 2020-04-21

- Please read this document carefully before using the product and observe the safety notices.

- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

INFORMATION

- ▶ New information regarding product safety and product recalls as well as the declaration of conformity can be obtained at oa@ottobock.com or from the manufacturer's service department (see inside or outside of back cover for addresses).
- ▶ You can request this document as a PDF file at oa@ottobock.com or from the manufacturer's service department (see inside or outside of back cover for addresses). The PDF file can also be displayed in a larger size.

These instructions for use will provide you with important information regarding the installation and proper use of the 476C00=SK350 Evolight wheelchair seat cushion.

All users and/or their attendants must be trained by qualified personnel in the use of the product with the aid of these instructions for use. In particular, users and/or attendants must be informed of the residual risks with the aid of the safety notices.

Please note the following:

- It may be necessary to adapt the product according to the needs of the user and the medical conditions. Adjustments may only be made by qualified personnel and in close coordination with the user.
- The manufacturer reserves the right to make technical changes to the model described in these instructions for use.

2 Product description

The wheelchair seat cushion is used as decubitus prophylaxis. The seating surface consists of a foam base with a cube structure. This cube structure contributes to improved pressure redistribution and the reduction of shear forces.

Two layers of spacer fabric (one layer above the cube structure, one layer sewn into the cover fabric) ensure very good air circulation. The horizontal ventilation channels facilitate the dissipation of moisture. This considerably improves the sitting climate.

The foam base is entirely enclosed in a cover. This cover is made of elastic, liquid-repellent, PU-coated fabric.

3 Intended use

The safe use of the product can be ensured only if it is used as intended in accordance with the information contained in these instructions for use.

3.1 Indications for use

The product is intended **exclusively** as a wheelchair seat cushion for adults with intact skin who use a wheelchair on a temporary or permanent basis.

The optimally suited seat cushion must be selected by trained, qualified personnel depending on the size and individual sitting needs of the user. It may only be used with a wheelchair in the appropriate size.

The allowed user weight is **35 kg [77 lbs] to 125 kg [275 lbs]**. We assume no liability if the product is used outside the allowed user weight range.

3.2 Indications

- Minor to pronounced or complete restrictions of mobility

- Decubitus risk

3.3 Contraindications

3.3.1 Absolute Contraindications

- None known

3.3.2 Relative Contraindications

- Failure to meet physical or mental requirements

3.4 Further Usage Restrictions

The product is designed for use by **one person** only. Re-use of the product is not permitted due to hygiene considerations.

3.5 Qualification

This product may only be selected and adapted by qualified personnel. Compliance with all manufacturer specifications and all applicable legal provisions is required. Please contact the manufacturer's service department for further information (see inside or outside of rear cover for addresses).

4 Safety

4.1 Explanation of warning symbols

 WARNING	Warning regarding possible serious risks of accident or injury.
 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

4.2 General safety instructions

 **WARNING**

Improper handling of packaging materials

Risk of suffocation due to neglect of the duty to supervise

- Packaging materials must be kept out of the reach of children.

 **CAUTION**

Altered centre of gravity resulting from use of the seat cushion

Risk of tipping over or falling due to unstable settings

- After installing the seat cushion in the wheelchair, always check the user's sitting position and the wheelchair's centre of gravity. The sitting position and centre of gravity must be appropriate for the intended use and the user's requirements. Work with the user to find the optimal sitting position.
- If necessary, adjust the position of the seat and axle to improve stability. Alternatively assess the suitability of anti-tip devices.

 **CAUTION**

Lack of testing prior to long-term use

Risk of skin reddening or pressure sore formation due to incorrectly selected seat cushion

- Any seat cushion should be tried out by the user under medical supervision for several hours prior to long-term use. The skin should be monitored for redness and/or other damage.

4.3 Safety instructions for use

WARNING

Improper handling of ignition sources

Risk of severe burns due to seat cushions, padding and upholstery catching on fire

- ▶ The seat cushions, padding and covers fulfil the normative requirements for flame resistance. However, they may still ignite if fire is handled improperly or negligently.
- ▶ Keep away from all ignition sources, especially lit cigarettes and open flames.

WARNING

Exceeding the service life

Serious injuries due to failure to observe the manufacturer's requirements

- ▶ Using the product beyond the specified expected service life leads to increased residual risk.
- ▶ Observe the specified service life.

WARNING

Independent modification of settings

Serious injuries to the user due to improper changes to the product

- ▶ Do not modify the settings established by the qualified personnel. Only the settings described in the section "Use" in these instructions for use may be adjusted independently.
- ▶ In case of problems with the settings, please contact the qualified personnel who adjusted your product.

CAUTION

Skin damage

Skin damage or pressure points due to overloading

- ▶ Check your skin for intactness before and during use of the product.
- ▶ Pay attention to diligent skin care and pressure redistribution during interruptions in using the product.
- ▶ If skin damage or other problems occur during use, stop using the product. Consult the qualified personnel.

CAUTION

Failure to observe care instructions

Risk of pressure sores due to incorrect skin care

- ▶ All seat cushions were developed by the manufacturer to reduce pressure on the buttocks. A seat cushion alone, however, cannot eliminate sitting pressure or possible pressure sores entirely.
- ▶ When using the seat cushion, always pay attention to correct skin care – especially through a healthy diet, hygiene and regular pressure relief.

CAUTION

Extreme temperatures

Hypothermia or burns due to contact with components, failure of components

- ▶ Do not expose the product to any extreme temperatures (e.g. direct sunlight, sauna, extreme cold).
- ▶ Do not leave the product in the immediate vicinity of heaters.

⚠ CAUTION

Use of the product during diagnostic examinations and therapeutic treatment

Impairment of the examination results or the effectiveness of treatment due to interactions of the product with devices that are used

- ▶ Make sure that examinations and treatments are carried out exclusively under the prescribed conditions.

NOTICE

Improper use

Risk of damage to the cushion surface due to user error

- ▶ Do not allow the wheelchair seat cushion to come into contact with sharp objects. This also applies to animals with sharp claws such as pet cats.
- ▶ Do not expose the wheelchair seat cushion to excessive heat or open flame.
- ▶ If the wheelchair seat cushion is expected to come into contact with liquid, such as spilt drinks or episodes of incontinence, always use it in conjunction with a liquid-repellent cover.
- ▶ Only use the official Ottobock incontinence covers for the product. The incontinence covers have been optimised for the wheelchair seat cushion to maintain the pressure redistribution capability of the seat cushion. Contact your prescribing therapist, clinician or specialist dealer to obtain a spare Ottobock cover.

NOTICE

Wear of seat cushions, padding and upholstery

Loss of functionality due to prohibited continued use

- ▶ Have the seat cushions, padding and upholstery replaced promptly in case of damage.

4.4 Nameplate

Label	Meaning
	A Additional information (optional) B Manufacturer's logo C Manufacturer's reference number D Manufacturer's product name E Global Trade Item Number (DI)¹⁾ F Textile care instructions with symbols according to ISO 3758 G Textile labelling H Manufacturer information/address I Symbol for medical device J WARNING! Read the instructions for use before using the product. Observe important safety-related information (e.g. warnings, precautions). K CE marking L Manufacturing date²⁾ M LOT or serial number (PI)³⁾ incl. barcode N Seam O Sewn in, not visible: LOT or serial number Manufacturer's reference number

¹⁾ UDI-DI to GS1 standard; UDI = Unique Device Identifier, DI = Device Identifier

²⁾ YYYY = year of manufacture; MM = month of manufacture; DD = day of manufacture

³⁾ UDI-PI to GS1 standard; UDI = Unique Device Identifier, PI = Product Identifier

4.5 Side effects

The following side effects may occur during use of the product:

- Circulatory disorders, risk of pressure sores

Contact a doctor or therapist in case of problems.

5 Delivery

5.1 Scope of delivery

INFORMATION

- ▶ To order spare parts, please contact the qualified personnel.
- ▶ When ordering spare parts, please specify the width and depth of your product. Check the sewn-in label on the cover for information on the dimensions.

The product is delivered preinstalled. The scope of delivery includes:

- Seat cushion (components see Page 12)
- Instructions for use

5.2 Storage

Store the product in a dry, enclosed room with sufficient air circulation and protection from external influences. Specific information about storage conditions: see Page 20.

6 Preparing the product for use

INFORMATION

- ▶ The procedures described in this section may only be carried out by qualified personnel.
- ▶ These instructions for use must be given to the user along with the product.

INFORMATION

- ▶ The seat cushion must be set up by qualified personnel before delivery.
- ▶ Before handing over to the user or attendant, perform sufficient seat tests under medical supervision. Have the seat cushion tested by the user as needed.

6.1 Setting Up

- 1) Take the seat cushion out of the packaging.
- 2) Ensure the cover fits well and there are no large folds or wrinkles on the top surface.
- 3) Position the seat cushion in the wheelchair. Make sure the side with the zip is at the rear and the hook-and-loop strap faces down.

7 Use

7.1 Positioning

CAUTION

Damage to the product, unexpected noise or odours

Health problems for the user due to improper handling

- ▶ Stop using the product immediately in the event of defects or other hazards that can result in personal injury, or if there are noises or smells that greatly differ from the delivered condition of the product.
- ▶ Contact the qualified personnel.
- ▶ Report damage to the product that may cause health problems for the user to Ottobock.

INFORMATION

- ▶ Place the seat cushion in the middle of the seat so that there is sufficient space on all sides for the wheels and frame.
- ▶ Optimum pressure redistribution is ensured when the leg support of the wheelchair is adjusted so that the thigh of the user completely rests on the seat cushion.

- ▶ Regularly remove dust and dirt particles from the hook-and-loop strap at the bottom so that the seat cushion remains firmly in place.

Correct positioning of the seat cushion in the wheelchair must be maintained for daily use:

- The foam insert's cube structure faces up.
- The spacer fabric lies above the cube structure and the rounded side faces to the rear.
- The hook-and-loop strap on the cover faces the wheelchair's seat bottom. It is used to attach the seat cushion in the wheelchair.

7.2 Removing/putting on the cover

Removing the cover

The cover can be removed for washing:

- 1) Unzip the cover completely.
- 2) Carefully pull off the cover along the foam base.
- 3) Remove the spacer fabric insert and the foam base.

INFORMATION: Cleaning information see Page 18.

Putting on the cover

- 1) Unzip the cover completely.
- 2) Insert the foam base into the cover. Make sure the cube structure faces up on the ready-to-use seat cushion, and the hook-and-loop strap on the cover faces down towards the wheelchair's seat bottom.
- 3) Set the spacer fabric insert onto the side of the foam base with the cube structure and smooth it with your hands, removing all wrinkles.
- 4) Carefully continue pulling the cover over the foam base until it is entirely enclosed by the cover.
- 5) Zip up the cover.
- 6) Ensure the cover fits well and there are no large folds or wrinkles on the top surface.

7.3 Cleaning

NOTICE

Improper cleaning of the wheelchair seat cushion

Risk of damage to seat cushion due to user error

- ▶ Do not machine wash the foam components.
- ▶ Do not dry the seat cushion and its components in the tumble dryer.
- ▶ Gently wipe the surface of the foam components with a soft, damp cloth or sponge; do not rub. Do not use a brush or similar on the surface.
- ▶ To prevent mould formation, the foam components must be completely dry before putting them into the cover.
- ▶ Do not wash the cover at a temperature higher than specified. The cover can shrink when washed at higher temperatures. This could impede the seat cushion's ability to redistribute pressure and damage the fabric.
- ▶ Do not use any bleaching agents or fabric softeners for cleaning.

INFORMATION

- ▶ In case of advanced wear and tear, have the components that are available as spare parts replaced by qualified personnel.
- ▶ To disinfect the cover, add a standard hygiene detergent during washing.
- ▶ Use an additional incontinence cover in case of heavy incontinence. Observe the care and cleaning instructions for the incontinence cover.

Clean the product every month.

7.3.1 Cleaning the cover

- 1) Unzip and remove the cover for separate washing (see Removing/putting on the cover).
- 2) Close the zipper on the cover before washing.
- 3) The cover can be washed at +60 °C [+140 °F] using a mild, environmentally friendly detergent. Recommendation: Wash the cover using the delicate cycle at +40°C [+104°F] to prevent excessive wear and tear.
4. Allow to air dry.

7.3.2 Cleaning the spacer fabric insert

- 1) Remove the spacer fabric insert from the cover for separate washing.
- 2) The insert can be washed at +60 °C [+140 °F] using a mild, environmentally friendly detergent. Recommendation: Wash the cover using the delicate cycle at +40°C [+104°F] to prevent excessive wear and tear.
- 3) Allow to air dry.

7.3.3 Cleaning the foam base

- 1) Remove the foam base from the cover to clean it (see Page 18).
- 2) Use lukewarm water for cleaning. If necessary, an alcohol-free disinfectant or mild detergent can be added.
- 3) Wipe the foam base with a damp cloth or sponge.
INFORMATION: When adding disinfectant or detergent, wipe the foam base with clear water in a second step.
- 4) Dry the foam base with a towel, dab or let air dry at room temperature.
- 5) Avoid exposure to direct heat (e.g. sun or heat from radiators and stoves).

8 Maintenance and repair

- The function of the product should be checked **before each use**.
- Check the product **monthly** for wear and tear as well as damage. The interval between inspections should be further reduced for very heavy users.
 - Check the cover for rips, holes and other damage.
 - Check the functionality of the zip.
 - Remove dust and dirt particles from the cover's hook-and-loop strap.
- The user is not permitted to make repairs to the product.
- Have damaged or worn parts replaced by qualified personnel as soon as possible.
- Servicing and repairs may only be carried out by qualified personnel or the manufacturer. This will ensure that only Ottobock spare parts are used for repairs.

9 Disposal information

Return the product to the qualified personnel for disposal.

All components of the product must be disposed of properly in accordance with the respective national environmental regulations.

10 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

10.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

10.2 Warranty

Further information on the warranty terms and conditions can be obtained from the qualified personnel that has fitted this product or the manufacturer's service (see inside back cover for addresses).

10.3 Service life

Expected lifetime: **2 years**

The design, manufacturing and requirements for the intended use of the product are based on the expected lifetime. These also include the requirements for maintenance, ensuring effectiveness and the safety of the product.

11 Technical data

Dimensions, weights, temperatures	
Dimensions (depth x width)	See label on the cover Minimum: 280 x 260 Maximum: 560 x 620
Seat cushion thickness [mm]	60
Weight ¹⁾ [kg]	0.85
User weight [kg]	35 – 125
Component	Material
Foam base	PUR foam core topped by spacer fabric made of 100% PES
Cover	Cotton/Trevira blend with spacer fabric (100% PES), washable

¹⁾ Reference size 400 x 400 mm, in ready-to-use condition, without incontinence cover

Ambient conditions

Temperatures and relative humidity	
Temperature during use [°C (°F)]	0 to +40 (+32 to +104)
Transport and storage temperature [°C (°F)]	0 to +40 (+32 to +104)
Relative humidity [%]	10 to 85; non-condensing

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A series of horizontal lines for writing, consisting of 20 lines.

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

A series of horizontal lines for writing, spanning the width of the page.

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Asia/Pacific

Otto Bock Australia Pty. Ltd.
Baulkham Hills NSW 2153
Australia
T +61 2 8818 2800
F +61 2 8814 4500
healthcare@ottobock.com.au

Beijing Otto Bock
Orthopaedic Industries Co., Ltd.
Beijing, 100015, P.R. China
T +8610 8598 6880
F +8610 8598 0040
news-service@ottobock.com.cn

Otto Bock Asia Pacific Ltd.
Kowloon, Hong Kong · China
T +852 2598 9772
F +852 2598 7886
info@ottobock.com.hk

Otto Bock HealthCare
India Pvt. Ltd.
Mumbai 400 021 · India
T +91 22 2274 5500 / 5501 / 5502
information@indiaottobock.com

Otto Bock Japan K. K.
Tokyo, 108-0023 · Japan
T +81 3 3798-2111
F +81 3 3798-2112
ottobock@ottobock.co.jp

Otto Bock Korea HealthCare Inc.
137-070 Seoul · Korea
T +82 2 577-3831
F +82 2 577-3828
info@ottobockkorea.com

Otto Bock
South East Asia Co., Ltd.
Bangkok 10900 · Thailand
T +66 2 930 3030
F +66 2 930 3311
obsea@ottobock.co.th

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Otto Bock Mobility Solutions GmbH