THUASNE

Malleo Dynastab® BOA®

fr	Chevillière d'immobilisation stabilisatrice avec système de serrage BOA® Fit System.....	6
en	Stabilising ankle-immobilising brace with BOA® Fit System closure.....	7
de	Stabilisierende Sprunggelenkorthese zur Ruhigstellung mit BOA® Fit System zum Festziehen.....	8
nl	Stabiliserende immobilisatiebrace voor de enkel met BOA® Fit System-klemsysteem.....	10
it	Cavigliera d'immobilizzazione stabilizzatrice con sistema di chiusura BOA® Fit System.....	11
es	Tobillera de inmovilización estabilizadora con sistema de ajuste BOA® Fit System.....	12
pt	Artelheira estabilizadora de imobilização com sistema de aperto BOA® Fit System.....	14
da	Immobiliserende ankelstøttebind med BOA® Fit System til stramning.....	15
fi	Immobilisoiva vakauttava nilkkatuki, jossa on BOA® Fit System -kiristysjärjestelmä.....	16
sv	Stabiliserande och immobiliserande ankelortos med fästsystemet BOA® Fit System.....	18
el	Σταθεροποιητική επιστραγαλίδα ακινητοποίησης με σύστημα σύσφιξης BOA® Fit System.....	19
cs	Stabilizační ortéza na kotník se stahovacím systémem BOA® Fit System.....	20
pl	Stabilizująca orteza skokowo-stopowa z systemem dopasowania BOA® Fit System.....	22
lv	Stabilizējošs imobilizācijas potītes stiprinājums ar BOA® Fit System savilkšanas sistēmu.....	23
lt	Stabilizuojantis imobilizuojantis čiurnos įtvartas su „BOA® Fit System“ suveržimo sistema.....	24
et	BOA® Fit System kinnitussüsteemiga stabiliseeriv fikseeriv hüppeliigese ortoos.....	26
sl	Imobilizacijska opornica za stabilizacijo gležnja s sistemom zategovanja BOA® Fit System.....	27
sk	Imobilizačná členková ortéza s upínacím BOA® Fit Systéмом.....	28
hu	Stabilizáló bokarögzítő BOA® Fit System szorítórendszerrel.....	29
bg	Обездвижаващ стабилизатор за глезен със система за затягане BOA® Fit System.....	31
ro	Glezniară stabilizatoare de imobilizare cu sistem de strângere BOA® Fit System.....	32
ru	Ортез стабилизирующий голеностопный с системой фиксации BOA® Fit System.....	33
hr	Steznik za stabilizaciju i imobilizaciju sa sustavom zatezanja BOA® Fit System.....	35
zh	带BOA® Fit System闭锁系统的稳定式护踝.....	36
ar	BOA® Fit System داعم كاحل لتحقيق الاستقرار والتثبيت مزود بنظام الشد.....	39

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

	0	16 - 19 cm	6 ^{3/4} - 7 ^{1/2} "
	1	19 - 22 cm	7 ^{1/2} - 8 ^{3/4} "
	2	22 - 26 cm	8 ^{3/4} - 10 ^{1/4} "
	3	26 - 30 cm	10 ^{1/4} - 11 ^{3/4} "

fr	Tour de cheville
en	Ankle circumference
de	Knöchelumfang
nl	Omvang van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Nilkanympärys
sv	Runt fotleden
el	Περίμετρος αστραγάλου
cs	Hlezenní stabilizační ortéza
pl	Obwód kostki
lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
sl	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferința gleznei
ru	Окружность голени
hr	Opseg gležnja
zh	脚踝尺寸
ar	محيط الكاحل

fr	Immobilisation	Stabilisation	Proprioception
en	Immobilization	Stabilization	Proprioception
de	Ruhigstellung	Stabilisierung	Propriozeption
nl	Immobilisatie	Stabilisatie	Proprioceptie
it	Immobilizzazione	Stabilizzazione	Propriocezione
es	Inmovilización	Estabilización	Propiocepción
pt	Imobilização	Estabilização	Proprioceção
da	Støtte	Stabilisering	Proprioception
fi	Immobilisoi	Stabiloi	Proprioseptio
sv	Immobilisering	Stabilisering	Proprioception
el	Ακιμντοποίηση	Σταθεροποίηση	Ιδιοδεκτικότητα
cs	Znehybnění	Stabilizace	Propriocepe
pl	Unieruchomienie	Stabilizacja	Propriocepcja
lv	Imobilizācija	Stabilizēšana	Propriocepcija
lt	Imobilizavimas	Stabilizavimas	Propriocepcija
et	Liikumatuks muutmine	Stabiliseerimine	Propriotseptsoon
sl	Imobilizacija	Stabiliziranje	Proprioceptivni dražljaji
sk	Imobilizácia	Stabilizácia	Propriocepcia
hu	Rögzítés	Stabilizálás	Propriocepció (helyzetérzékelés)
bg	Обездвижаване	Стабилизация	Проприоцепция
ro	Imobilizare	Stabilizare	Propriocepție
ru	Иммобилизация	Стабилизация	Проприоцепция
hr	Imobilizacija	Stabilizacija	Propriocepcija
zh	固定	穩定	本体感受
ar	التثبيت	الاستقرار	استقبال الحس العميق

fr

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en

STABILISING ANKLE-IMMOBILISING BRACE WITH BOA® FIT SYSTEM CLOSURE

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Textile parts: neoprene, polyamide, elastane, polyester, polyethylene, polyurethane.
Elastic straps: polyester, elastane.
Self-fastening tabs: polyamide.
Plastic parts: polyamide 66.
Side reinforcements: stainless steel.

Properties/Mode of action

Stabilising ankle brace to provide immobilisation, incorporating rigid stays and elastic straps to give stability and immobilisation to the ankle.
Fast, durable and micro-adjustable BOA® Fit System for a custom fit.
A slim product that will fit easily inside a shoe.
This is a bilateral product.

Indications

Moderate to severe ankle sprain.
Chronic ankle instability.
Ankle pain/oedema.
Sport activities resumption.

Contraindications

Do not use the product if the diagnosis has not been confirmed.
Do not apply the product in direct contact with broken skin.
Do not use in the event of known allergy to any of the components.
History of venous or lymphatic disorders.

Precautions

Verify the product's integrity before every use.
Do not use the product if it is damaged.
Choose the appropriate size to fit the patient, referring to the size chart.
It is recommended that a healthcare professional supervises the first application.
Strictly comply with your healthcare professional's prescription and recommendations for use.
For hygiene and performance reasons, do not re-use the product for another patient.
Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).
In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a health care professional.
Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).
Do not wear the product in a medical imaging machine.
Do not wear the product when driving a vehicle.
The systematic use of a long sock is recommended when wearing the device.
It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

If the product comes into contact with water, make sure to remove the lateral stays. Dry out the fabric part and wipe the stays thoroughly using a dry cloth.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters,etc.) or wounds of various degrees of severity.
Possible risk of venous thrombosis.
Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

We recommend that the product should be worn inside a shoe by daytime whilst walking, and by night for protection in order to prevent and limit sudden movements.
Before putting the ankle brace on, make sure the laces are completely unfastened.

A. Fitting the ankle brace:

Slide the foot inside the ankle brace. Ensure that the heel is resting inside the opening provided at the rear of the product for this purpose. ❶ ❷

B. Fastening the ankle brace:

Tighten the ankle brace using the BOA® Fit System closure system; to do this, push the wheel in until you hear a ‘click’, then turn clockwise until the correct level of support is obtained. ❸

C. Fitting the straps:

Take a strap in each hand, then pass them over each other across the upper foot. ❹
Wrap around the ankle, cross the two straps over again around the rear of the heel, and attach the self-fastening tabs to the upper portion of the ankle-brace fabric. ❺
Verify the secure grip of the self-fastening tabs over the fabric area by pressing on the self-fastening part. Do not place the self-fastening tabs over the black plastic portion. ❻

D. Releasing the ankle brace

Pull on the wheel until you hear the ‘click’ to unfasten the laces. ❼

E. Fitting inside a shoe

Once the ankle brace is in position, slide the foot carefully inside a closed shoe and fasten the shoe’s closure mechanism. ❶
In the event of any discomfort at the ankle-bone area, it is possible to change the position of the stays: Detach the self-fastening tabs from the black fabric area, take hold of the rigid metal part and remove it. Slide the stays above or below the stitching located either side of the ankle brace. ❷

Care/maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label.
Make sure to remove the lateral stays before washing. Replace in the original location before next use.
Close the self-fastening tabs and close the product (BOA® Fit System) before washing. If possible, use a washing net.
Dry in a ventilated area, away from sources of heat.

Storage

Store at room temperature, preferably in the original packaging.

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©Thuasne - 2043701 (2023-02)
Thuasne UK Ltd