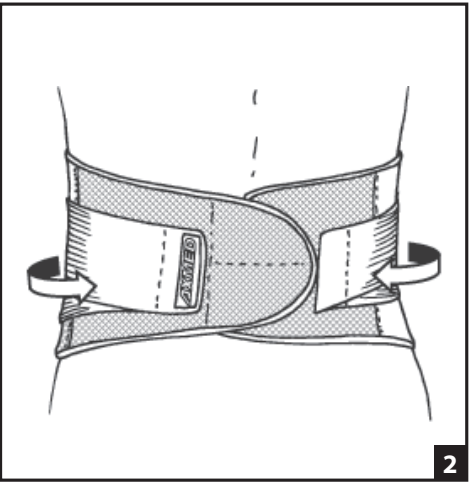
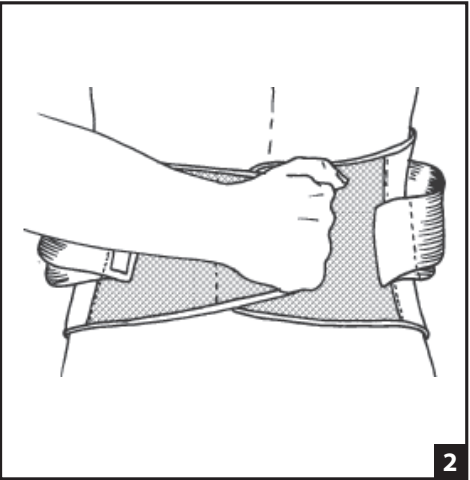
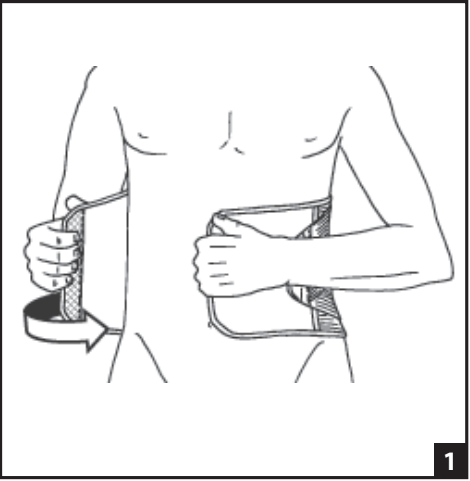
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE / INDICATIONS
Cure and prevention of acute or chronic lumbago.

CONTRA-INDICATIONS – PRECAUTIONS
For maximum efficiency, make sure to choose the proper size and adjust the side pulls to the required compression.

APPLICATION INSTRUCTIONS
1) Position the belt around the waistline and close using contact closure.
2) Pull the side pull elastic straps to the desired level of compression and affix them on the front of the belt using the contact closure tabs.

CARE INSTRUCTIONS
Hand wash in cold water with mild soap, rinse thoroughly. Air dry. Do not machine dry or bleach.

Composition: 40% Lycra - 25% Polyamide - 18% Cotton - 9% Latex - 8% Polyester

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

LATEX WARNING CONTAINS LATEX

PATIENT USE
INTENDED FOR USE ON A SINGLE PATIENT.

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GARANTI: DJO, LLC vil reparere eller erstatte hele