Ally® Rings

B | BRAUN

en	recyclable
de	recyclelbar
es	reciclable
fr	recyclable
it	riciclabile
nl	recyclebaar
bg	рециклируем
cs	recyklovatelné
hu	újrahasznosítható
lv	pārstrādājams
pl	do recyklingu
pt	reciclável
ru	перерабатываемый
sk	recyklovateľné
vi	có thể tái chế
th	รีไซเคิลได้
fi	kieränettävä
sr	reciklirajuće
ro	reciclabil
	ätvernningsbar

	B. Braun Avitum AG Schwarzenberger Weg 73-79 34212 Melsungen Germany
	Zhende Medical Co., Ltd. Gaobu Town, 312035, Shaoxing, Zhejiang, CHINA
	Shanghai International Holding Corp. GmbH(Europe) Eiffestraße 80, 20537 Hamburg, GERMANY

Australian Sponsor:

B. Braun Australia Pty Ltd Level 5, 7-9 Irvine Place Bella Vista NSW 2153
Made in China
V25.01.20

en Instruction for use

1. Product description: The ostomy ring is intended to fill in cavities/folds in the intact skin around the stoma, for use with colostomies, ileostomies, urostomies, it can help to protect the skin.

2. Composition: Made of release liner and hydrocolloid (CMC, medical hot-melt glue and pectin).

3. Specifications
Size: ø48mm, Thickness: 2,2mm
Size: ø48mm, Thickness: 3mm
Size: ø48mm, Thickness: 4,5mm
Size: ø98mm, Thickness: 3mm

4. Intended purpose: Ostomy ring can be used to fill in or caulk uneven skin contours near the stoma to create a flatter surface and help prevent ostomy drainage from getting under the ostomy baseplate. They may be cut, stretched, or stacked together to help improve the fit of a pouching system.

5. Intended population: The product may be used on individuals with a colostomy, ileostomy or urostomy.

6. Intended user: Health care professionals, caregivers and patients in a hospital or home environment.

7. Indications: For use with a colostomy, ileostomy or urostomy.

8. Warnings:
8.1 Single Use.
8.2 For external use only.
8.3 Do not apply to wounds or skin ulcers.
8.4 Should skin irritation occur, contact your doctor or another health professional.
8.5 Do not use on individuals with a known sensitivity to the ingredients.
8.6 Do not use on individuals with impaired immune systems or people receiving immune suppressive treatment.
8.7 Do not use on premature babies, or infants receiving incubator care.
8.8 Do not use if the product expires.

9. Information for use:
9.1 Preparation:
Ensure that the hands and skin around the stoma is clean and dry before applying.
9.2 Application:
9.2.1 Apply the ostomy ring to the skin around the stoma.
9.2.2 Adjust the shape again if needed.
Mold the ring with your fingers so that the edge of the ostomy ring is securely smoothed onto the skin, or directly paste the ostomy ring on the surface of the baseplate.
9.2.3 Smooth gently with wet fingers if necessary for better adhesion to skin.
9.2.4 Align the center of the baseplate with the opening to ensure a good fit.

10. Method of removal: Removed with lukewarm water. A small amount of residue can be left on the skin when applying the next application. It is better to leave a slight residue than risk irritating the skin through complete removal.

11. Storage: Keep away from sunlight. Store products at cool and dry conditions – do not freeze.

12. Disposal: The product should be disposed of together with household waste. Do not flush the product down the toilet.

13. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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